JAY[®]

J3 CUSHION

AIR INSERT

IMPORTANT CONSUMER INFORMATION

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NOTICE: This owner's manual is a supplement to the J3 Cushion Owner's Manual. The J3 Cushion owner's manual should be referenced in conjunction with this owner's manual prior to the installation, set-up or use of this product.

NOTICE: This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end-user.

SUPPLIER: This manual must be given to the user of this product

USER: Before using this product, read this entire manual, and the J3 Cushion manual, and save the instructions for future reference.

GENERAL WARNINGS

WARNING

CHECK INFLATION AT LEAST ONCE A DAY!

WARNING

UNDER-INFLATION: DO NOT use an under-inflated cushion. Using a cushion that is under-inflated reduces or eliminates the cushion's benefits, increasing the risk to the skin and other soft tissue. If your cushion appears under-inflated or does not appear to be holding air, check to make sure that all valve(s) are closed by turning the inflation valve(s) clockwise. If your cushion is still not holding air, contact your clinical caregiver, supplier immediately.

WARNING

OVER-INFLATION: DO NOT use an over-inflated cushion. Using an over-inflated cushion will not allow you to sink into the cushion and will reduce or eliminate the cushion's benefits increasing risk to skin and other soft tissue.

WARNING

CUSHION AND COVER ORIENTATION: Product must be used with the air cells facing up.

WARNING

PRESSURE: Changes in altitude may require adjustment to your cushion. Check your cushion when changing altitude of 1,000 feet (300 meters) or greater.

WARNING

PUNCTURE: Keep your cushion away from sharp objects which may puncture the air cells and cause it to deflate.

WARNING

HEAT/FLAME: DO NOT expose your cushion to high heat, open flames or hot ashes.

WARNING

CLIMATE: If a cushion has been in temperatures less than 32°F / 0°C and experiences unusual stiffness, allow the cushion to warm to room temperature, then open air valve. Roll up the pad and then unroll it until the neoprene is soft and pliable again. Then repeat proper adjustment instructions before using.

WARNING

DO NOT use on top or in conjunction with another cushioning product other than the J3 Cushion with which it was supplied.

WARNING

OBSTRUCTIONS: DO NOT place any obstructions between the user and the cushion, as it will reduce product effectiveness.

WARNING

HANDLING: DO NOT use any valve as a handle for carrying or pulling the cushion.

WARNING

PRODUCT USE: DO NOT use your cushion as a water floatation device (e.g., a Life Preserver).

CAUTION– SOLUTIONS: DO NOT ALLOW OIL-BASED LOTIONS OR LANO-LIN TO COME IN CONTACT WITH YOUR CUSHION, AS THEY MAY DEGRADE THE MATERIAL.

CAUTION– OZONE GENERATORS: THE US ENVIRONMENTAL PROTECTION AGENCY, HEALTH CANADA AND OTHER GOVERNMENT AGENCIES HAVE ISSUED WARNINGS REGARDING THE USE OF OZONE GENERATORS. PROLONGED EXPOSURE TO OZONE WILL DEGRADE RUBBER, FABRICS AND OTHER MATERIALS USED IN THE MANUFACTURE OF THE PRODUCT AND MAY AFFECT THE PERFORMANCE OF THE PRODUCT AND VOID THE PRODUCT WARRANTY.

CAUTION– DO NOT USE A PUMP OR REPAIR KIT OTHER THAN THOSE PROVIDED BY SUNRISE MEDICAL. DOING SO MAY VOID YOUR PRODUCT'S WARRANTY.

CLEANING AND DISINFECTING YOUR J3 AIR INSERT

Cleaning and disinfecting are two separate processes. Cleaning must precede disinfection.

CLEANING THE AIR INSERT

To clean the Air Insert, gently pull it out of the recess in the foam base, taking care to gently pull the valve through the foam. Deflate the cushion, close the valve(s) and place in a large sink. Gently scrub the air cells and the spaces between them with a soft plastic bristle brush, sponge or washcloth. Liquid dish-washing or laundry detergent or other general-purpose cleaning product is suggested. For institutional cleaning, a general house-keeping disinfectant cleaner may be used.

DISINFECTING THE AIR INSERT

To disinfect the Air Insert, use a germicidal product that meets infection control needs. A disinfectant with tuberculocidal properties is recommended. Follow the manufacturer's dilution directions. For home disinfection, use solutions of approximately 1/2 cup liquid household bleach per quart of water (125 ml bleach to 1 liter of water). Keep the clean cushion wet with bleach solution for 10 minutes. Rinse thoroughly and dry.

WARNING

Disinfectants are not effective on porous surfaces, such as foam, if the cushion becomes soiled or contaminated. DO NOT use with multiple users.

CAUTION— DO NOT USE OIL-BASED LOTIONS OR LANOLIN ON YOUR CUSHION AS THEY MAY COMPROMISE THE INTEGRITY OF THE NEOPRENE RUBBER.

REPAIRS

Each J3 Cushion with a Air Insert is shipped complete with a repair kit and instructions for minor repairs. The following information can be used if repair instructions have been lost or misplaced.

If A Leak Occurs

1. Place your cushion cells down and over-inflate until all four corners touch.
2. Immerse the over-inflated cushion in water to locate the leak.
3. Once the leak has been located, mark the leak by inserting a toothpick into the hole
4. Allow the cushion to dry thoroughly.
5. Remove the toothpick, and clean the area around the hole with the alcohol wipe provided. Let the surface dry.
6. Peel the backing from the patch provided and, centering the patch, place it over the hole.
7. Press the patch down firmly until a good seal is achieved.
8. The cushion is now ready for use. Adjust the cushion for proper inflation following the instructions in this manual.

Although patches do not change the performance of the cushion, they should be monitored periodically to ensure effectiveness in addressing the leak. If the patch is not successful in addressing the problem, or if more extensive damage is present, please contact your supplier immediately.

Replacing Valves

1. Firmly grip the rubber valve stem and remove the old inflation valve by using pliers.

CAUTION— DO NOT REMOVE AN INFLATION VALVE BY HOLDING THE CUSHION AND PULLING THE VALVE, DOING SO MAY TEAR THE CUSHION.

2. Moisten the ribbed end of the replacement valve and insert it into the valve stem until the rubber is snug to the body of the valve.

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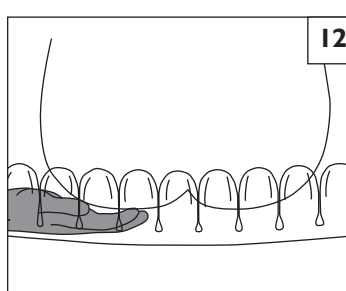
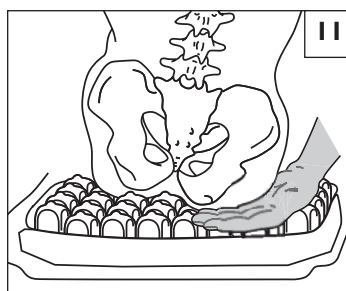
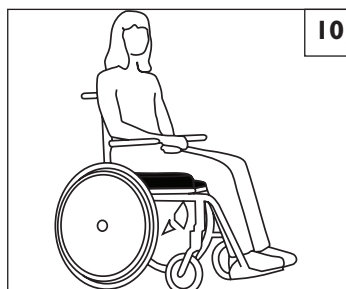
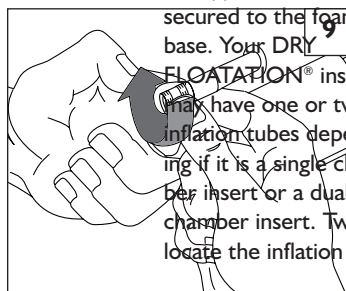
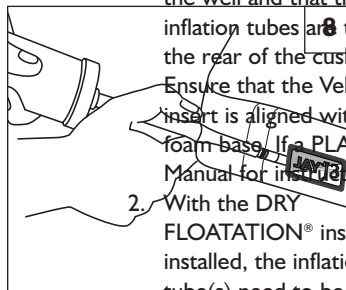
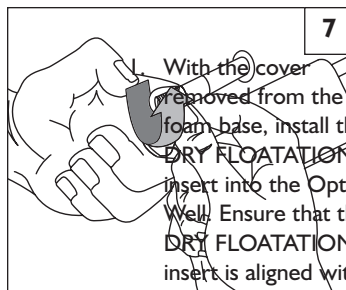
1. With the cover removed from the J3 foam base, install the DRY FLOATATION® insert into the Opti-Well. Ensure that the DRY FLOATATION® insert is aligned with the well and that the inflation tubes are to the rear of the cushion. Ensure that the Velcro attached to the DRY FLOATATION® insert is aligned with the matching Velcro adhered to the foam base. If a PLA insert is being used, see the J3 Owner's Manual for instructions on its installation. (Fig. 3)

2. With the DRY FLOATATION® insert installed, the inflation tube(s) need to be secured to the foam base. Your DRY FLOATATION® insert may have one or two inflation tubes depending if it is a single chamber insert or a dual chamber insert. Two channels on the foam base are used to locate the inflation tube(s). (Fig. 4)

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Sunrise Medical AB
Sunrise Medical B.V.