***Jay Foam Cushion Products***  
*User Instruction Manual & Warranty*

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## INSTRUCTIONS FOR USE FOR THE JAY® BASIC - 300 SERIES MODELS, JAY COMBI - 200 SERIES MODELS, JAY SOFT COMBI - 2200 SERIES MODELS, AND JAY TRIAD™ - 1200 SERIES MODELS

### Caution

Installing a cushion on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards potentially resulting in injury. Always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability. Your Jay foam cushion was designed for sitting comfort and postural support. Users should inspect their skin daily for redness. The clinical indicator for tissue breakdown is skin redness. If your skin develops redness, discontinue the use of the cushion immediately and see your doctor or therapist. If you need more pressure distribution for skin protection, ask your authorized supplier for a free demonstration of any of the other Jay cushion products.

**Note:** To ensure the proper use of your cushion, please read these instructions and keep them for future reference.

### Fitting a Jay foam cushion

Select a Jay foam cushion that matches the width of your wheelchair. The cushion length should extend to within 1" - 2" (2.5 - 5cm) from the back of the knees. Place the cushion to the back of the chair with the material identification tags to the back and the bump towards the front. The product label should be visible on the right side of the cushion. Adjust the footrests so your legs rest comfortably on the cushion. Lowering the footrests on your wheelchair may help reduce the pressure on your ischials (seat bones) and coccyx (tailbone).

**Note:** Proper footrest adjustment may enhance sitting comfort. Also, avoid using the cushion on overly stretched seat upholstery. Replace stretched seat upholstery or purchase a Jay solid seat insert for your cushion. Contact a authorized supplier for a free demonstration.

### Fitting a lumbar support

If your Jay foam cushion includes a lumbar support, its proper use may help to reduce lower back discomfort and sitting fatigue.

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1. Adjust lumbar support height so that maximum thickness of lumbar support is at your waist or belt line.
2. Attach the lumbar support by looping one Velcro® strap attachment over the back of the chair, under the push handle and back around the front of the chair. The hook Velcro on the back of the strap should secure to the loop Velcro on the front of the strap.
3. Repeat the attachment process on the opposite side, making sure that the lumbar support is centered on the wheelchair.
4. If additional support is required, a 1/2" (1 cm) thick foam insert can be placed inside the cover behind the lumbar support. This acts as a shim to increase the thickness of the lumbar support.  
**Note:** Junior size lumbar supports attach with an elastic strap and buckle instead of Velcro® straps.  
**Hint:** Some people find that varying the height of the lumbar support during the course of the day helps relieve sitting fatigue. They also find that placing the cushion about 1" (2.5 cm) further forward in the wheelchair makes it more comfortable to use with the lumbar support. If your cushion did not include a lumbar support but you would like its benefits, contact an authorized supplier for a free demonstration.

#### Easy maintenance

Monthly cleaning and regular maintenance may help extend the life of your cushion. During cleaning, component inspection is recommended. Check the cover for tears and excessive wear.

#### To clean cover

Remove the cover from the base. Turn the cover inside out and machine wash in warm water and drip or tumble dry on low heat. No ironing required.

**Note:** Do not dry clean covers or use industrial washers and dryers to clean cover. Do not steam autoclave. Do not bleach.

After washing, fit cover back over foam base. Ensure the back of the cover matches up to the back of the foam base.

**Note:** If utilized in an institutional setting, write the resident's name with a permanent marker on the content label of the covers. This will help to ensure the return of the covers after washing.

Use only Jay-designed covers in conjunction with Jay foam cushions.

#### To clean foam base

Remove the cover from the foam base. Wipe with warm water and soap. Wipe off with a clean cloth. Avoid complete submersion of the foam base in water.

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**ENGLISH****DUTCH****SWEDISH****FINNISH****DANISH****LIMITED WARRANTY**

Sunrise Medical guarantees its products to be free from defects in materials and workmanship for a specific period of time from date of purchase. Determine which cushion was purchased and use the following information to determine which warranty applies.

- The Jay lumbar support is guaranteed for one year from date of purchase.
- The Jay Basic Cushion is guaranteed for one year from date of purchase.
- The Jay Triad Cushion is guaranteed for two years from date of purchase.
- The Jay Combi Cushion and Jay Soft Combi Cushions are guaranteed for five years from date of purchase. Sunrise Medical will, at its option, repair or replace your cushion without charge. The warranty does not apply to, punctures, tears, or burns, nor to the cushion's cover. Claims and repairs should be processed through the nearest authorized equipment supplier. Except for express warranties made herein, all other warranties including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded. There are not warranties which extend beyond the description on the face hereof. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

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**Sunrise Medical AB**