JAY® Fit Seating

IMPORTANT CONSUMER INFORMATION

NOTICE: This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end user.

SUPPLIER: This manual must be given to the user of this product.

USER: Before using this product, read this entire manual and save for future reference.

WARRANTY REGISTRATION: Visit www.SunriseMedical.com to register this product.

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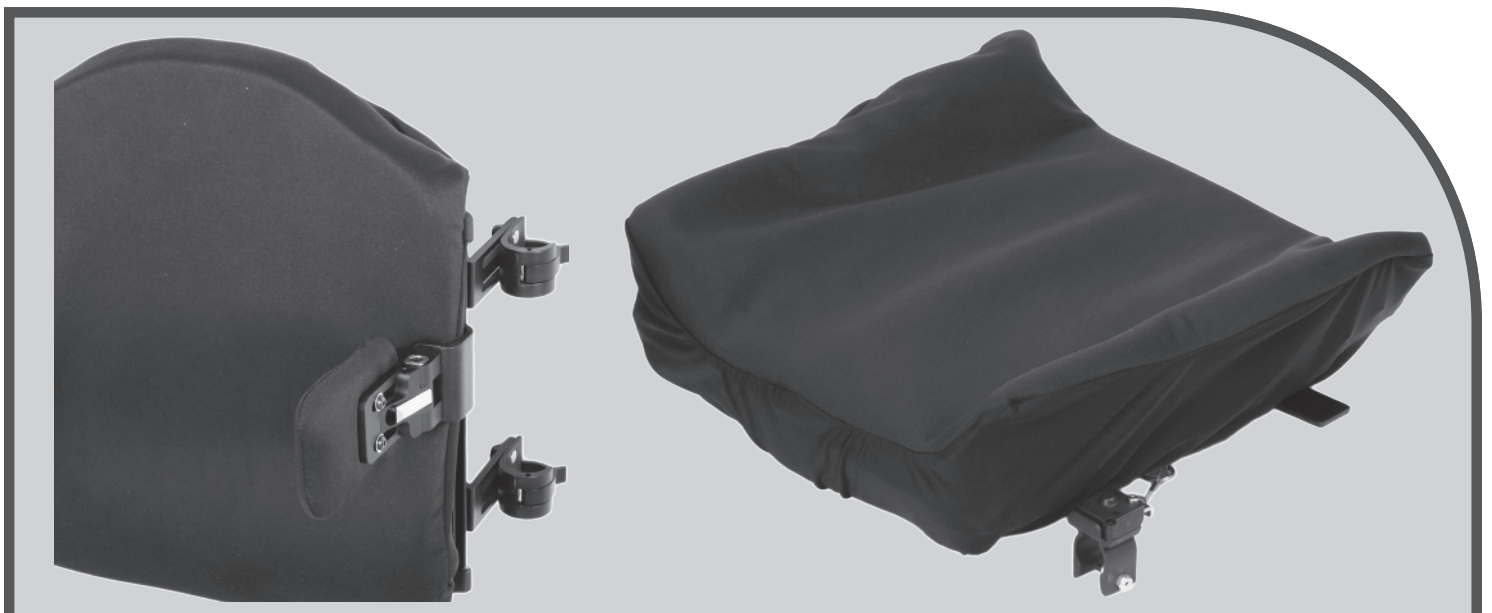
Manuel d'instructions

SUNRISE MEDICAL LISTENS

Thank you for choosing the JAY® Fit Seating System. We want to hear your questions or comments about this manual, the safety and reliability of your product, and the service you receive from your Authorized Sunrise Supplier. Please feel free to write or call us at the address and telephone number below:

**Sunrise Medical (US) LLC
Customer Service Department
12002 Volunteer Blvd.
Mount Juliet, TN 37122, USA
(800) 333-4000**

Be sure to return your warranty card, and let us know if you change your address. This will allow us to keep you up to date with information about safety, new products and options to increase your use and enjoyment of this Fit system. If you lose your warranty card, call or write and we will gladly send you a new one.

FOR ANSWERS TO YOUR QUESTIONS

Your Authorized Dealer knows your product best, and can answer most of your questions about safety, use and maintenance. For future reference, fill in the following:

Supplier: _____

Address: _____

Telephone: _____

Serial #: _____ Date/Purchased: _____

ADDITIONAL INFORMATION YOU SHOULD KNOW

No component of this product was made with Natural Rubber Latex.

DISPOSAL AND RECYCLING INFORMATION

When this product reaches the end of its life, please take it to an approved collection or recycling point designated by your local or state government. This product is manufactured using a variety of materials. Your product should not be disposed of as ordinary household waste. You should dispose of your product properly, according to local laws and regulations. Most materials that are used in the construction of this product are fully recyclable. The separate collection and recycling of your product at the time of disposal will help conserve natural resources and ensure that it is disposed in a manner that protects the environment.

Ensure you are the legal owner of the product prior to arranging for the product disposal in accordance with the above recommendations

TABLE OF CONTENTS

- I. Read before use.....2
- II. Introduction.....3
- III. General Warnings.....5
- IV. Use and Maintenance7
- V. Installation & Adjustment.....9
- VI. Accessories.....10
- VII. Warranty.....11

A. CHOOSE THE RIGHT CHAIR & SAFETY OPTIONS

This product is intended for single person use only. Selection of the type of support systems, options, and final adjustment rests solely with the user and their health care professional. Choosing the best product and set-up for safety and health depends on such things as:

1. Type of disability, strength, balance and coordination.
2. The types of hazards that must be overcome in daily use.
3. The need for options that will improve your positioning, safety and comfort (such as positioning belts, or special seating systems).

B. REVIEW THIS MANUAL OFTEN

Before using this system, you, and each person who may assist you, should read this entire manual and make sure to follow all instructions. Review the warnings often, until they are second nature to you.

C. WARNINGS

The word “**WARNING**” refers to a hazard or unsafe practice that may cause **severe injury or death** to you or to other persons. The “Warnings” are in Three main sections, as follows:

1. **V- GENERAL WARNINGS**
A safety checklist and a summary of risks you need to be aware of before you use this equipment.
2. **VI- SAFETY WARNINGS: FALLS & TIP-OVERS**
Learn about the safe use of your equipment, and how to avoid a fall or tip-over while performing daily activities.
3. **VII- WARNINGS- COMPONENTS & OPTIONS**
Learn about the components and options that can be selected for safety and Positioning needs. Consult your Authorized Supplier and your Health care advisor to help you choose the best set-up and options for safe use.

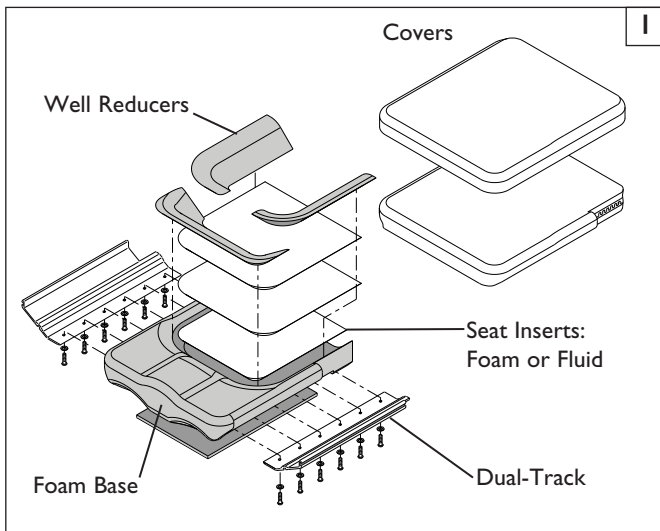
NOTE - Where they apply, you will also find “Warnings” in other sections of this manual. Heed all warnings in this section. If you fail to do so a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

Sunrise Medical recommends that a clinician such as a doctor or therapist experienced in seating and positioning be consulted to determine if the JAY® Fit seating is an appropriate solution. These seating systems should only be installed by an Authorized Sunrise Medical dealer.


JAY® Fit Seating system

The JAY® Fit Seating system is designed to provide the important benefits of a proper back support for a wide variety of users. It is designed to provide simple, comfortable and effective support to help maximize function and increase sitting tolerance. In addition, it is light-weight, durable and easy to upgrade.


JAY® Fit Seating

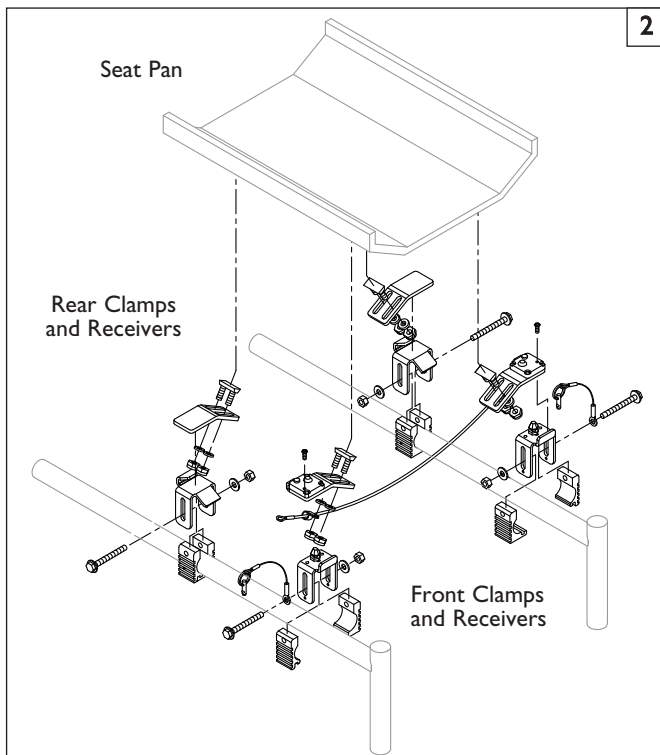
JAY® Fit Height Adjustable Seating Hardware

JAY® Fit Back support

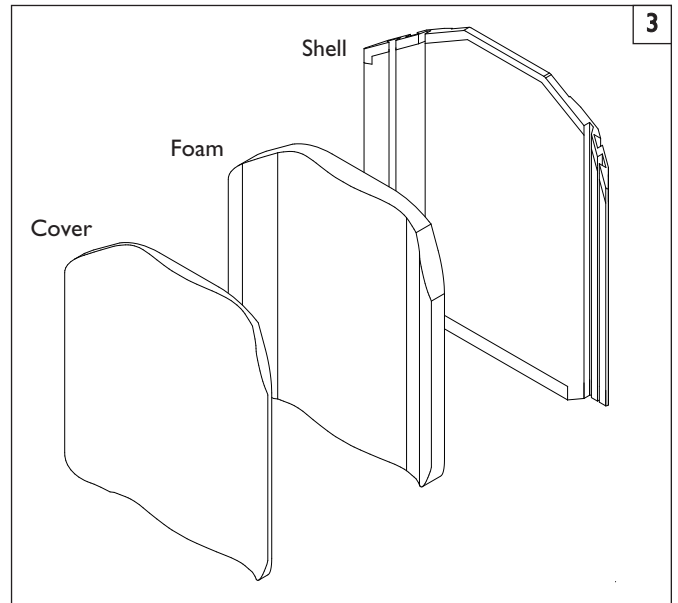
JAY® Fit TH back hardware

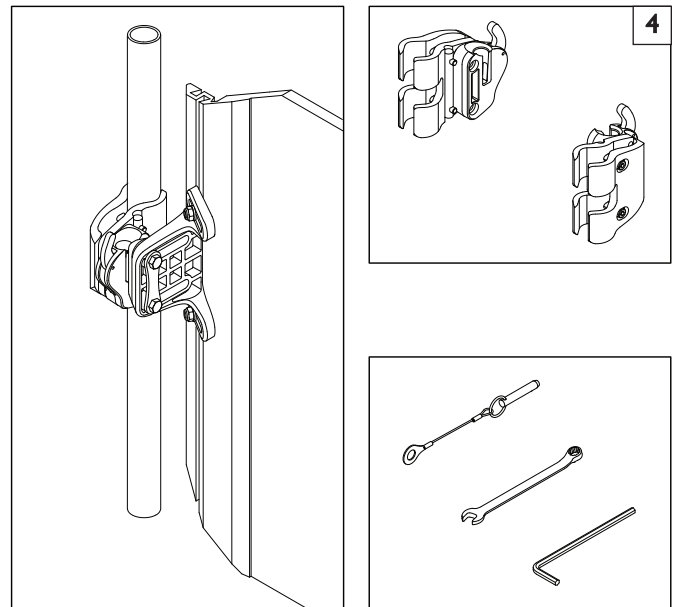
Locking pin, hex key, and 10mm wrench

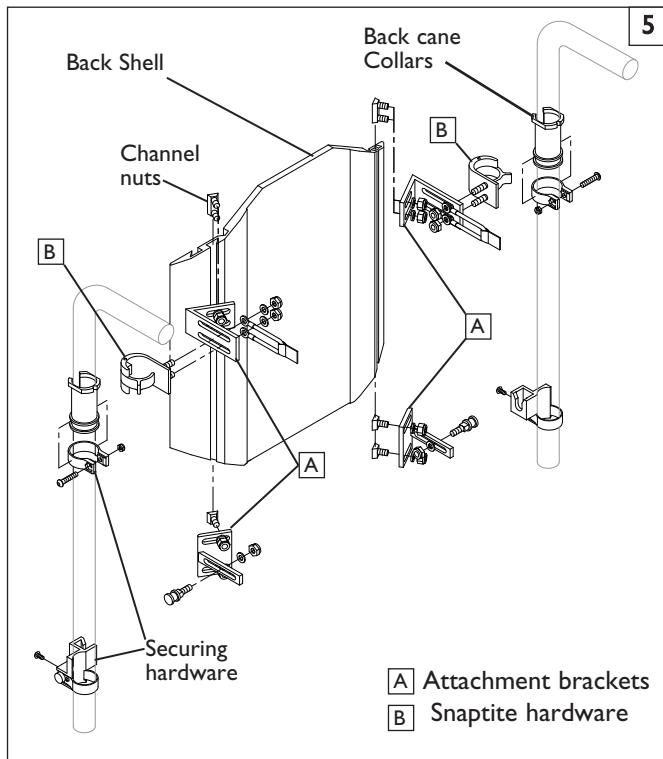
JAY® Fit Back Hardware: Adjustable Quick Mount

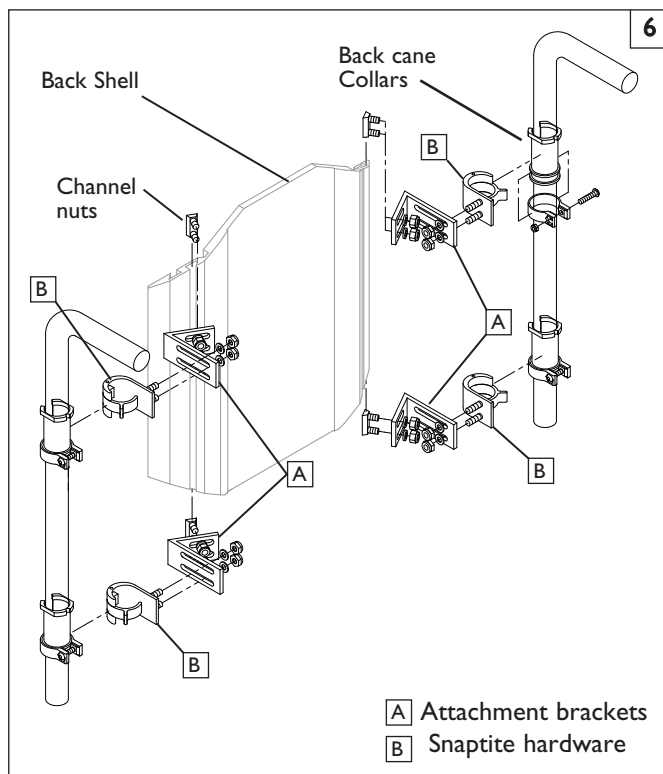JAY® Fit Back Hardware: Snaptite with collars

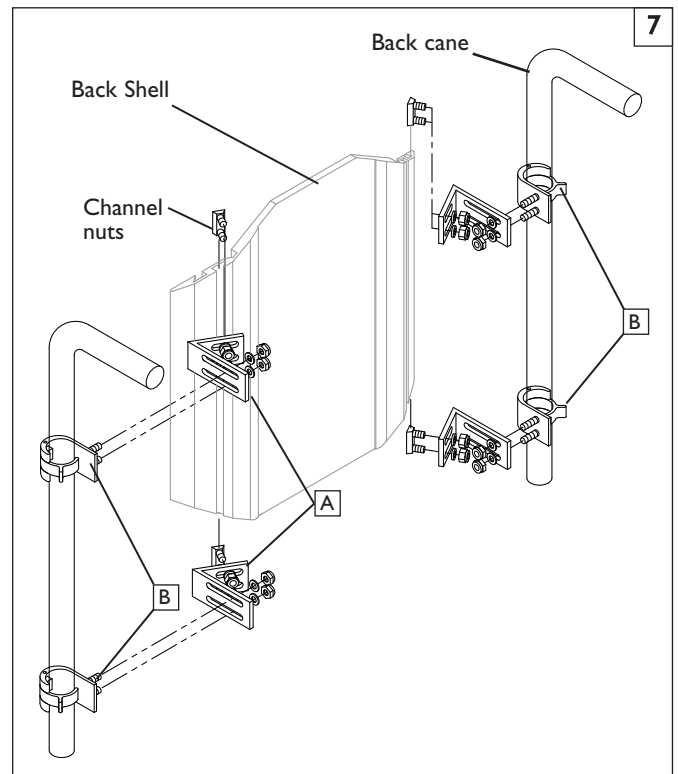JAY® Fit Back Hardware: Snaptite no collars

A. WEIGHT LIMITS**⚠ WARNING**

NEVER exceed the weight limit specified by Sunrise Medical. The weight capacity provided is for the combined weight of a rider and items carried using on-board storage. If you do exceed the weight limit, damage to your product, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

Weight Capacities

JAY™ Fit Seating - 250lbs/(113kg).

B. INTENDED USE

JAY™ Fit seating is intended use is to provide support and comfort to persons limited to a seated position.

⚠ WARNING

DO NOT use this system for purposes other than what is intended by the manufacturer. If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

C. ATTENDANTS AND CAREGIVERS**⚠ WARNING**

Before you assist a rider, be sure to read all warnings contained in this manual, and follow all instructions that apply. Be aware that after consulting a healthcare advisor, you will need to learn safe and proven body mechanics to use and create assistive methods best suited to your abilities.

D. ACCESSORIES**⚠ WARNING**

Unauthorized modifications or use of parts, or accessories not supplied or approved by Sunrise Medical may change the chair structure. This will void the warranty and may cause a safety hazard.

Some problems that may occur, but are not limited to:

1. Adding a component to the frame, changing the structural integrity of the chair.
2. Any modification or disassembly can potentially create an unsafe situation where rider and/or attendant are put at risk.

E. REDUCE THE RISK OF AN ACCIDENT**⚠ WARNING**

1. BEFORE riding, you should be trained in the safe use of this chair by your health care advisor.
2. Practice bending, reaching and transfers until you know the limit of your ability. Have someone help you until you know what can cause a fall or tip-over and how to avoid doing so.
3. Be aware that you must develop your own methods for safe use best suited to your level of function and ability.
4. NEVER try a new maneuver on your own until you are sure you can do it safely.
5. Get to know the areas where you plan to use your chair. Look for hazards and learn how to avoid them.

F. SAFETY CHECKLIST**⚠ WARNING****Before Each Use Of Your Seating system:**

1. Make sure the attachment (mounting hardware is secure and tight. Check for noise, vibration, or a change in ease of use. (They may loose fasteners, or damage to your chair).
2. Immediately repair any problem. Your authorized supplier can help you find and correct the problem.
3. If your chair has anti-tip tubes, lock them in place.

If you fail to heed these warnings, damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

G. CHANGES & ADJUSTMENTS**⚠ WARNING**

1. See your authorized supplier and have them adjust seating components any time a change or adjustment needs to be made.
2. Unauthorized modifications or use of parts not supplied or approved by Sunrise may change the chair structure. This will void the warranty and may cause a safety hazard.

If you fail to heed these warnings, damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

H. CENTER OF GRAVITY**⚠ WARNING**

Installing a back support on a wheelchair may affect the center of gravity of the wheelchair, and may cause the wheelchair to tip backwards potentially resulting in injury to the rider.

I. ANTI-TIPS**⚠ WARNING**

Anti-tip tubes can help keep your chair from tipping over backward in normal conditions.

1. Sunrise recommends the use of anti-tip tubes
2. ALWAYS assess the need for anti-tips that could be added to the wheelchair to help increase the stability of the wheelchair and seating system.

J. PUSHING OR LIFTING**⚠ WARNING**

Do not use the JAY™ Fit back to move or push the wheelchair.

1. Do not use the back handle to lift the wheelchair. It is intended only for Back removal.
2. Improper use of the back by pushing or pulling may cause the back to unexpectedly detach from the wheelchair.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

K. COMPATIBILITY**⚠ WARNING**

The JAY™ Fit is designed to be compatible with most wheelchairs with the following exception:

1. Wheelchairs that are angle adjustable with recline or tilt, and result in a back angle greater than 60° from the vertical

Back use with these types of wheelchairs may cause the back to accidentally disconnect from the wheelchair. This could result in a fall or potential injury of the rider and/or caregiver.

L. POSITIONING BELTS (Optional)

WARNING

Use positioning belts ONLY to help support the rider's posture. Improper use of these belts may cause severe injury to or death to the rider.

1. Make sure the rider does not slide down in the wheelchair seat. If this occurs, the rider may suffer chest compression or suffocate due to pressure from the belts.
2. The belts must be snug, but must not be so tight that they interfere with breathing. You should be able to slide your open hand, flat, between the belt and the rider.
3. A pelvic wedge or a similar device can help keep the rider from sliding down in the seat. Consult with the rider's doctor, nurse or therapist to find out if the rider needs such a device.
4. Use positioning belts only with a rider who can cooperate. Make sure the rider can easily remove the belts in an emergency.
5. NEVER Use Positioning Belts:
 - a. As a patient restraint. A restraint requires a doctor's order.
 - b. On a rider who is comatose or agitated.
 - c. As a motor vehicle restraint. In an accident or sudden stop the rider may be thrown from the chair. Wheelchair seat belts will not prevent this, and further injury may result from the belts or straps.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

M. CUSHIONS

WARNING

1. JAY™ Fit cushions are designed for the relief of most, but not all sitting pressure.
2. If you suffer from pressure sores or if you are at risk that they will occur, you may need a special seat system or a device to control your posture. Consult your doctor, nurse or therapist to find out if you need such a device for your well-being.
3. Seat pans are not intended to be used as a direct seating surface. A cushion or other seating surface should be placed on the seat pan before use.

N. FASTENERS

WARNING

Many of the screws, bolts and nuts on this chair are special high-strength fasteners. Use of improper fasteners may cause your chair to fail.

1. ONLY use fasteners provided by an authorized supplier (or ones of the same type and strength, as indicated by the markings on the heads).
2. Over or under-tightened fasteners may fail or cause damage to chair parts.
3. If bolts or screws become loose, tighten them as soon as you can.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

O. PINCH POINTS

WARNING

When hinged or adjustable components are moved or adjusted, there is a potential for a pinch point to exist. To avoid pinch points:

1. Use an open Palm to lock or fasten hardware and components.
2. Always be aware of pinch point areas and use caution.

P. POSTURAL SUPPORT ACCESSORIES

WARNING

When using Removable, adjustable, or hinged components:

1. Check the function of all moveable component accessories that may be added to this device.
2. Ensure that these items are fully adjusted and locked into position to provide the necessary support to the user.
3. Items such as swing-away laterals, abductors, and head supports are an important to the safety and health of the user and must be checked regularly to make sure they have not moved or been left unlocked accidentally.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

Q. ENVIRONMENTAL CONDITIONS

WARNING

Contact with water or excess moisture may cause your chair to rust or corrode. Avoid all extreme weather situations if possible.

1. Do not use your components in a shower, pool or other body of water. The parts are not water-tight and may rust or corrode from the inside.
2. Avoid excess moisture (for example, do not leave your chair in a damp bathroom while taking a shower).
3. Dry your components as soon as you can if they get wet, or if you use water to clean them.

R. JAY™ FIT SEATING TRANSIT SAFETY

WARNING

JAY™ Fit seating has been dynamically tested for use in a motor vehicle. This system conforms with ANSI-RESNA WC-20. If JAY™ Fit seating is to be used in a transit situation, it must be mounted to a wheelchair that conforms to the performance requirements of ANSI/RESNA WC-19. For Rider safety, Please follow all installation, use, and maintenance instructions within this manual as well as the transit instructions listed below .

1. If possible and feasible, the rider should transfer to the Original Equipment Manufacturer vehicle seat and use the vehicle restraints.
2. The distance between the top of the user's shoulder and the top of the back should not exceed 6.5" (16.5 cm).
3. The wheelchair must be labeled as appropriate for use as a seat in a motor vehicle, dynamically tested to the performance requirements of ISO Standard 7176-19 and installed, used and maintained according to the manufacturer's instructions.
4. If the original wheelchair seat was replaced, the new seat must be approved for wheelchair transit, installed and used as indicated in the manufacturer's instructions.
5. The Wheelchair Tiedown and Occupant Restraint System (WTORS) should be compatible with the specific wheelchair, used as indicated in the manufacturer's instructions and should comply with the performance requirements of ISO Standard 10542.
6. The Wheelchair must be forward facing during transport.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others

⚠ WARNING

The owner of this equipment is responsible for making sure that it has been set up and adjusted by an Authorized Dealer under the advice of a healthcare professional. The back and hardware may require periodic safety and function checks. Adjustments should be performed by your Authorized Dealer. Always use parts and/or accessories that have been recommended or approved by Sunrise Medical.

A. MAINTENANCE AND CLEANING

1. Proper maintenance and cleaning will improve performance and extend the useful life of your equipment.
2. Clean your equipment regularly. This will help you find loose or worn parts and make your equipment easier to use. You will need a mild detergent solution and cleaning rags.
3. If discovered, have loose, worn, bent or damaged parts replaced **before using the seating system.**

B. CRITICAL SAFETY CHECKS

1. Fasteners:

Sunrise Medical recommends that all fasteners be checked for wear, such as loose bolts or broken components **Every 6 months.** Loose fasteners should be retightened according to the installation instructions. Contact your Authorized Dealer immediately to replace any components. Do not continue to use the system after identifying loose or broken components.

C. EASY CLEANING

1. Back and Cushion cover

- a. Remove the cover from backshell by detaching the Velcro®. Unzip the zipper on the bottom of the cover.
- b. Remove the foam and turn the cover inside out.
- c. Zip the cover closed to avoid snagging.
- d. Machine wash in warm water (50°C).
- e. Drip and tumble dry in on Low Heat.

NOTE – Do not dry clean covers or use industrial washers or dryers to clean covers. Do not steam autoclave. Do not bleach. Laundry care symbol definitions are shown.

2. Back shell

- a. The backshell should be wiped clean with soap and water on a regular basis to prevent buildup of dirt and grime.


3. Positioning components

- a. Remove the components one at a time, noting the placement for re-assembly.
- b. Positioning components can be wiped clean with soap and water.
- c. **DO NOT** immerse positioning components underwater.


Laundry Care Symbols		8
	Machine wash in warm water (50° C)	
	Only non-chlorine bleach, when needed	
	Drip dry	
	Do not iron	
	Do not dry clean	

D. WHEELCHAIR DIMENSIONS 9 10

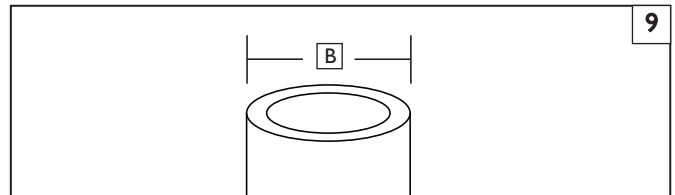1. The JAY™ Fit is designed to replace the wheelchair’s sling upholstery. Prior to installation determine if the wheelchair used has compatible back canes, by measuring the width between your back canes as seen in (Fig. 5).
2. Next check that the back canes are a compatible diameter. The JAY™ Fit hardware can be mounted on wheelchair back canes ranging from 3/4" to 1 1/8". Measure your back cane diameter (Fig. 6) and refer to Table 2 to ensure your back canes are compatible.
3. If you cannot determine that the back canes are compatible please contact your authorized Sunrise Medical provider or Sunrise Medical customer service.

JAY Fit Width (all contours)	Cane diameter (B)			
	Minimum width		Maximum width	
	inch	cm	inch	cm
10" - 18"	3/4	1.9	1 1/8	2.85

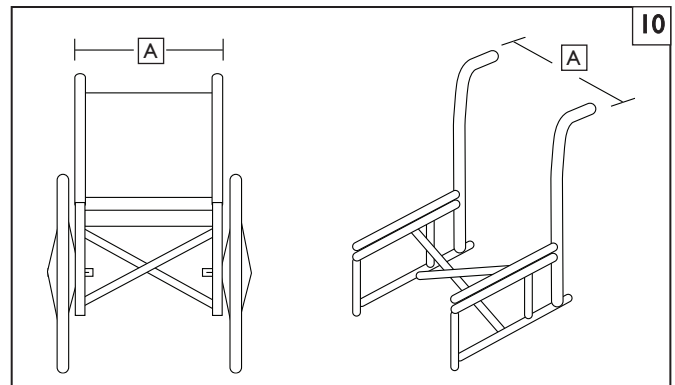JAY Fit Back Width* (all contours)	Width of back canes from outside to outside of the tubes (A)			
	Minimum width		Maximum width	
	inch	cm	inch	cm
10"	9.7	24.6	12.3	31.2
12"	11.7	29.7	14.3	36.3
14"	13.7	34.9	16.3	41.3
16"	15.7	40.0	18.3	46.4
18"	17.7	45.1	20.3	54.5

A. REMOVAL AND REPLACEMENT TH HARDWARE 11

After installation/adjustment, the back can be easily removed from the wheelchair if necessary.

1. Removal:

- a. Push both release levers(A) forward to unlock the hardware.
- b. Once both levers are in the forward position, lift the back straight up, using the handle located on the top of the back.

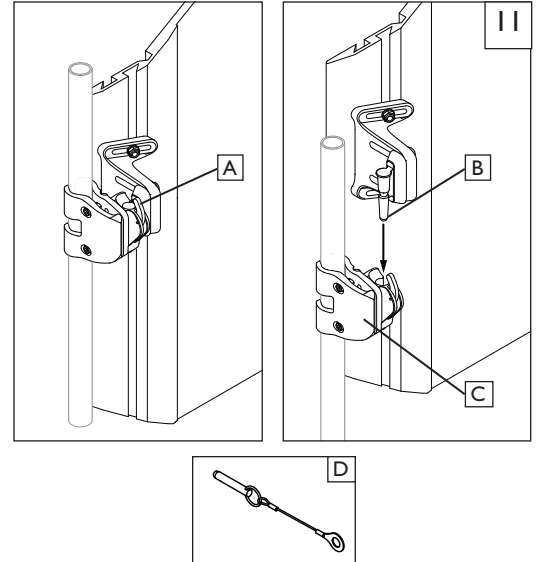
2. Proper re-attachment after removal

- a. To re-attach the back and hardware receivers, line up the pins(B) to the receivers.
- b. Push the back into place on the receivers(C).
- c. If properly inserted, the pins will “click” into a locked position.

3. Locking Pins

The locking pins provide a safety feature which prevents the back from being released unintentionally from the wheelchair.

- a. To lock, insert the pins (D) into the mounting receivers (C).
- b. Remove the pins if the quick-release capability is desired.

⚠ WARNING

If you are uncertain that you have heard the locking “click”, pull up on the back using the handle to test and make sure it has been securely locked.

If you fail to heed this warning damage to your equipment, a fall, or loss of control may occur and cause severe injury to the rider or others.

B. REMOVE OR INSTALL HEIGHT ADJUSTABLE SEATING 12 13 14

After installation/adjustment, by the authorized supplier, the seating can be easily removed from the wheelchair if necessary.

1. Removal:

- a. Remove the locking pin(E) from both receivers(F) on the seating system.
- b. Pull on the lanyard (G) under the seat to unlock the hardware.
- c. Lift the seat from the frame of the chair.

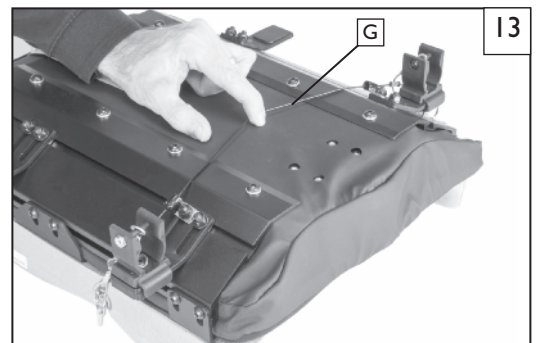
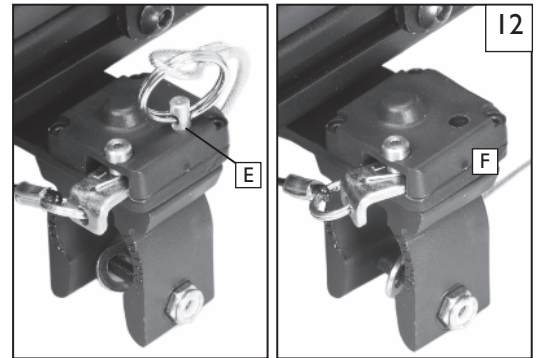
2. Proper installation

- a. To re-attach the seat, line up the pins(H) to the receivers(I).
- b. Push the seat into place on the receivers.
- c. If properly installed, the pins will “click” into a locked position.

3. Locking Pins

The locking pins provide a safety feature which prevents the seat from being released unintentionally from the wheelchair.


- a. To lock, insert the pins (E) into the mounting receivers (F) once the seating it attached.
- b. Remove the pins if the quick-release capability is desired.

⚠ WARNING

ALWAYS be sure to insert the locking pin when you are finished with the installation of the seat or backrest. Test it to make sure it has been securely locked.

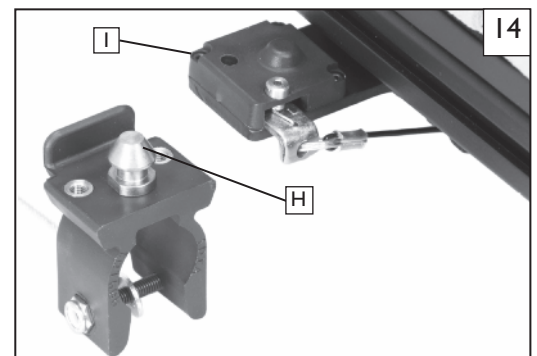If you fail to heed this warning damage to your equipment, a fall, or loss of control may occur and cause severe injury to the rider or others.

C. HEIGHT AND WIDTH ADJUSTMENT FOR TH HARDWARE 15

Please read the following instructions before beginning the installation. To install the JAY™ Fit with 2-point hardware, it is best to begin without a user in the wheelchair. Once installed, the user can be seated back in the wheelchair for minor adjustments.

NOTE – A higher location on the back canes are generally better.

1. Height adjustment

- a. Using the 4mm hex key, Loosen the clamp screws (A)

NOTE – For Large tube diameters, the clamp screws may have to be removed completely.

- b. Adjust and align the receivers(B) to the wheelchair back canes at the desired height.
- c. Hand tighten the clamp screws (A).
- d. The receivers may require additional lateral adjustments to ensure proper alignment.

2. Width adjustment 16

- a. Using a 10mm wrench loosen the bracket nuts(C).
- b. Adjust the brackets evenly in or out so that the pins are aligned with the receivers on the backcanes. Lightly tighten the bracket nuts.
- c. Once the hardware is at the optimum location, and symmetrical, tighten nuts to (100 in.-lbs.).




3. Attaching and aligning the back shell. 17

- a. Attach the backshell to the hardware receivers by inserting left and right mounting pins (D) in the left and right receivers (E).
- b. Locate the back to the approximate location, relative to both the chair and the user.
- c. Tighten bolts on the inside of the mounting pin (D). The recommended torque specification is 100-110 in/lbs. (11.3-12.4 Nm)
- d. Once all components have been properly aligned, tighten the clamp screws(A) to approximately 75-85 in-lbs (8.5-9.3 Nm).
- e. Finish by inserting the pin into the receiver and releasing a few times to ensure smooth operation.

4. Locking Pins

The locking pins provide a safety feature which prevents the back from being released unintentionally from the wheelchair.

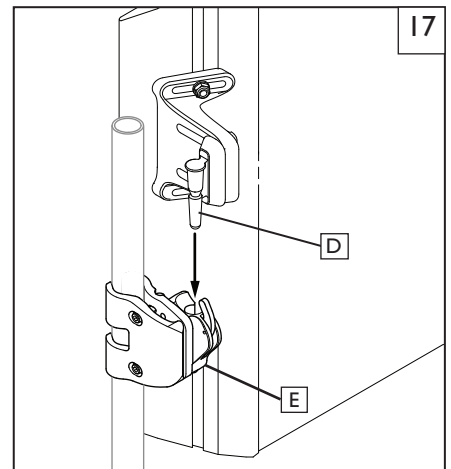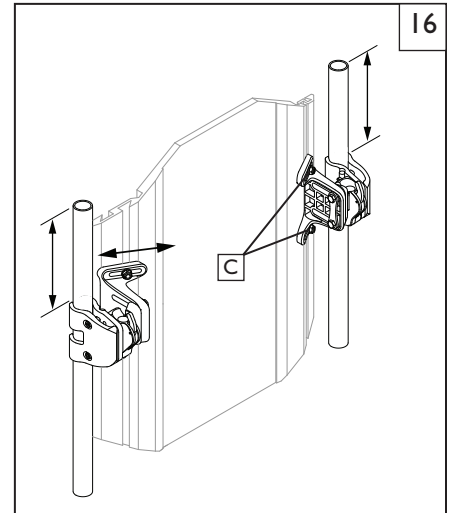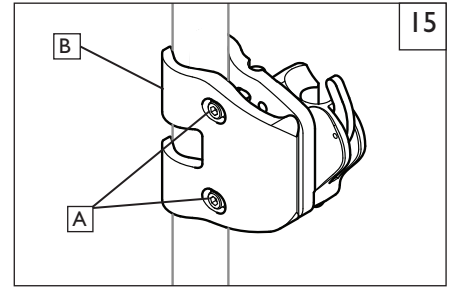- a. To lock, insert the pins into the mounting receivers.
- b. Remove the pins if the quick-release capability is desired.

A. POSITIONING COMPONENTS FOR SEATING

Well Reducers and cushion inserts

B. SPINE-ALIGN POSITIONING COMPONENTS

Spine Align Positioning Components are designed to provide intimately shaped support within an “off-the-shelf” back. Depending on your market, Spine Align components may be included with your back.

I. Spine Align Positioning Component installation

- a. Have the user remove pressure from the Back by leaning or sliding partially forward if possible.
- b. Release Velcro strip on the top cover flap and lift the cover forward to reveal the inside of the JAY[®] Fit Back.
- c. Place the positioning component(s) on the Velcro inside of the shell and reattach the top cover flap. Pull down making certain that the top seam is aligned with the top of the shell. If seam does not line up, the bottom of the cover may need to be loosened.
- d. Have the user test the current Spine Align setup for comfort and support and repeat steps a-c until adjustment is complete.
- e. Multiple configurations and combinations of Spine Align components can be used. If additional Spine Align shapes or sizes are required, please contact your dealer or Sunrise Medical at 800-333-4000.
- f. Mounting an optional headrest.
The JAY[®] Fit system back has a universal headrest plate mounted to it. Care should be taken to use the proper screw length when mounting.
- g. Secure the foam and cover to the shell.
To secure the back foam and cover on the Fit System back, slide the top edge of the cover over the top lip of the shell. Press the back in place against the shell, pull the two button snaps under the bottom of the shell and snap to the buttons.

NOTE - If you require further information on the JAY[®] Fit Spine Align system, please review our animation at www.SunriseMedical.com.

24 MONTH LIMITED WARRANTY

1. Each JAY[®] Fit seating system is carefully inspected and tested to provide peak performance. Every JAY[®] Fit seating system is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the back's removable cover.
2. Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded. There are not warranties which extend beyond the description on the face hereof. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold..
3. For goods provided by Sunrise Medical Pty Ltd in Australia, our goods come with a guarantee by Sunrise that cannot be excluded under the Australian Consumer Law. You are entitled to replacement or refund for a major failure & for compensation for any other reasonable foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality & the failure does not amount to a major failure. The benefits to you given by this warranty are in addition to your other rights & remedies under a law in relation to the goods to which
4. If you have any questions: Contact Sunrise Medical Customer Service at 1-800-333-4000

Sunrise Medical S.r.l.
Via Riva, 20 – Montale
29122 Piacenza
Italia
Tel.: +39 0523 573111
www.SunriseMedical.it

Sunrise Medical AG
Erlenauweg 17
CH-3110 Münsingen
Schweiz/Suisse/Svizzera
Tel +41 (0)31 958 3838
www.SunriseMedical.ch

Sunrise Medical AS
Delitoppen 3
1540 Vestby
Norge
Telefon: +47 66 96 38 00
post@sunrisemedical.no
www.SunriseMedical.no

Sunrise Medical AB
Neongatan 5
431 53 Mölndal
Sweden
Tel.: +46 (0)31 748 37 00
post@sunrisemedical.se
www.SunriseMedical.se

MEDICCO s.r.o.
H – Park, Heršpická 1013/11d,
639 00 Brno
Czech Republic
Tel.: (+420) 547 250 955
Fax: (+420) 547 250 956
www.medicco.cz
info@medicco.cz
Bezplatná linka 800 900 809

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Sunrise Medical Australia
11 Daniel Street
Wetherill Park NSW 2164
Australia
Ph: +61 2 9678 6600
Email: enquiries@sunrisemedical.com.au
www.SunriseMedical.com.au

Sunrise Medical (US) LLC
North American Headquarters
12002 Volunteer Blvd.
Mount Juliet, TN 37122, USA
(800) 333-4000
(800) 300-7502
www.SunriseMedical.com

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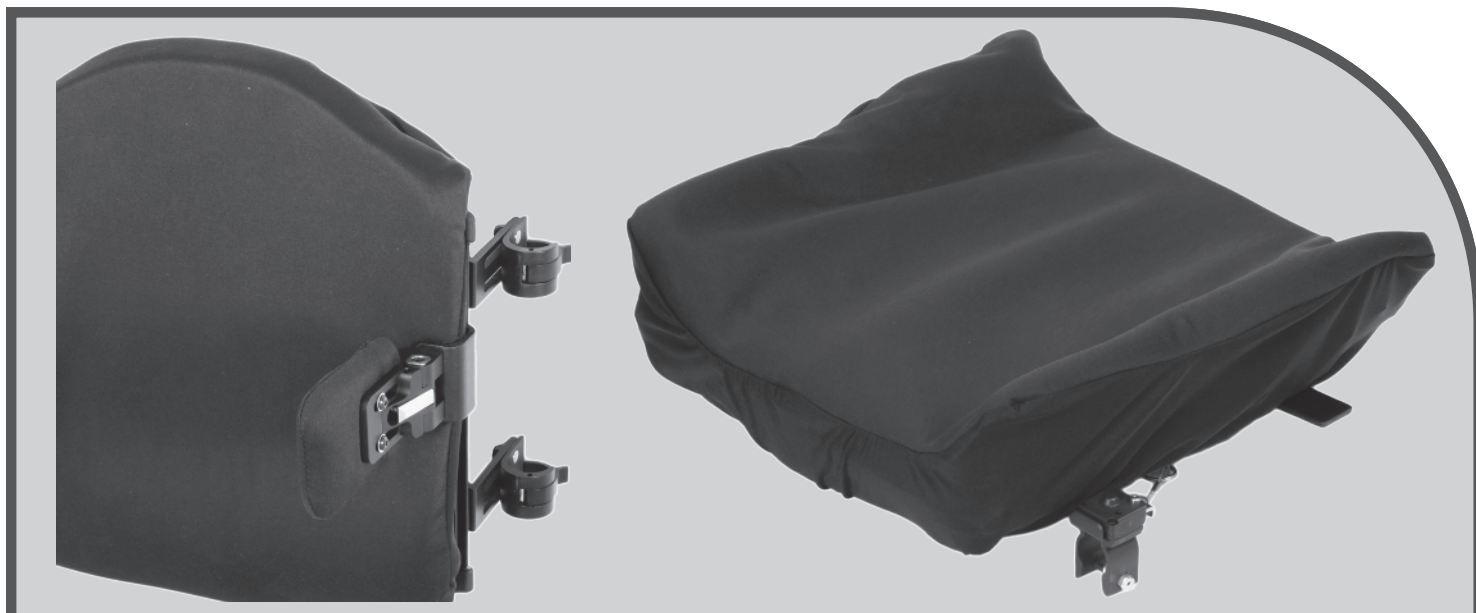
**Sunrise Medical (US) LLC
Customer Service Department
12002 Volunteer Blvd.
Mount Juliet, TN 37122, EE.UU.
(800) 333-4000**

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© 2025 Sunrise Medical (US) LLC
Sunrise Medical B.V.
Sunrise Medical HCM B.V.
Sunrise Medical S.A.S
Sunrise Medical Canada Inc.